




丹娜生物
DNM



*Detect and Distinguish the Virus
Prescribe the Right Medicine*

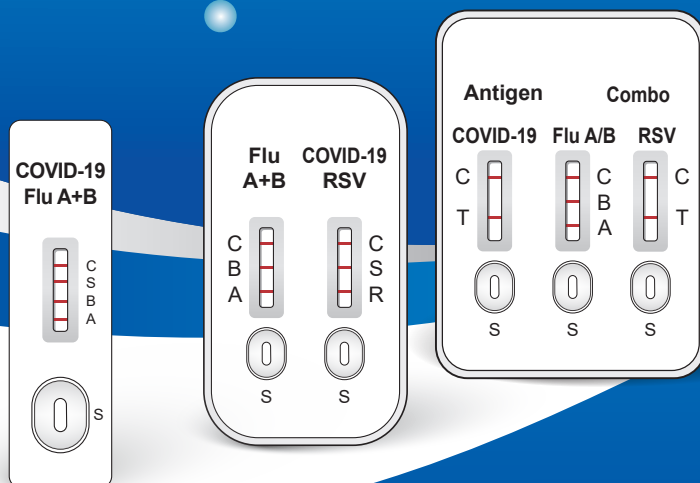

Dynamiker Influenza A+B Rapid Test
Dynamiker COVID-19/Influenza A+B Combo Rapid Test
Dynamiker Influenza A+B/RSV Ag Combo Rapid Test
Dynamiker COVID-19/Influenza A+B/RSV Ag Combo Rapid Test

COVID-19/Flu/RSV viruses are causing waves of respiratory illness among many countries across the continents. By the coming of winter, people stay indoors more, gathering more and contact closer, this increases the exposure and infections of COVID-19/Flu/RSV virus.

The symptoms of those viruses infections are similar. However, the treatments and Isolation guidance are different. So it's meaningful to know which virus you have.

Is it Covid-19, Flu, or RSV?

There are three respiratory illnesses are spreading in the winter. Here are the symptoms:

Signs and symptoms	RSV	Flu	Covid-19
Symptom onset	Gradual	Sudden	Gradual
Muscle and body aches	Sometimes	Common	Sometimes
Cough	Common	Common	Common
Diarrhea	Rare	Sometimes	Sometimes
Fatigue	Sometimes	Common	Common
Fever	Sometimes	Common	Common
Loss of taste or smell	Rare	Rare	Sometimes
Shortness of breath	Sometimes	Rare	Common
Runny or stuffy nose	Common	Common	Common
Sneezing	Sometimes	Sometimes	Sometimes
Sore throat	Sometimes	Sometimes	Common
Headache	Sometimes	Common	Common

Quote from "A medical worker collects a swab sample for a Covid-19 test in Los Angeles on Aug. 5, 2022." Original By Aria Bendix

"I don't think anybody would ever go, 'Hey, listen, I think you have a virus based on your symptoms,' and feel confident to say what virus that is," said Dr. Frank Esper, a pediatric infectious disease specialist at Cleveland Clinic.

The only way to know for certain, through, is to get tested.

Dynamiker Tests


They are intended to detect and differentiate the SARS-CoV-2 virus, Influenza A and Influenza B virus, and Respiratory syncytial virus.

- **Dynamiker Influenza A+B Rapid Test**
- **Dynamiker COVID-19/Influenza A+B Combo Rapid Test**
- **Dynamiker Influenza A+B/RSV Ag Combo Rapid Test**
- **Dynamiker COVID-19/Influenza A+B/RSV Ag Combo Rapid Test**

Specification


Specification	20 tests per kit
Method	Colloidal Gold Immunochromatography
Specimen	Nasopharyngeal/Oropharyngeal swabs
Storage and transfer	2-30 °C
Time to report	15 mins

Operation Flow Chart

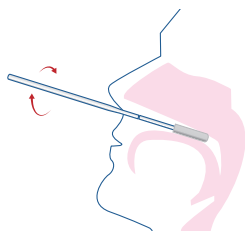


1

Insert the swab into both nostrils of the patient. The swab tip should be inserted up to 2.5 cm (1inch) from the edge of the nostril. Roll the swab 5 times along the mucosa inside the nostril to ensure that both mucus and cells are collected.

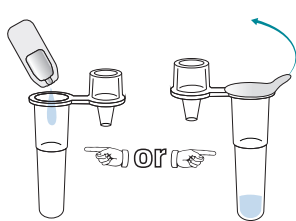


2

Twist off the head of the buffer, dispense the buffer into the extraction tube; (If your buffer is integrated in the extraction tube, unseal the buffer).

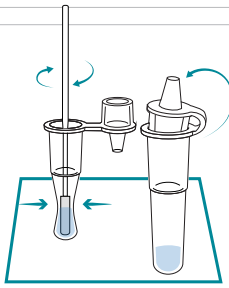


3

Insert the swab into the extraction tube containing the extraction buffer; Rotate the swab at least 6 times while pressing the head against the bottom and sides of the extraction tube; Remain the swab in the extraction tube for 1 minute, then remove it. The extracted solution will be used as a test sample.



4

Carefully refer to the instruction for use prior to performing the test; Take out the kits 30 mins before test, ensure that tests and samples are at room temperature; Place test cassettes on flat and clean bench; add 2 or 3 drops unknown extracted samples into sample pad;

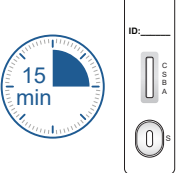

5

Read and record the results after 15 minutes (No longer than 20 minutes). Abnormal results may occur after 20 minutes.

Dynamiker Biotechnology(Tianjin)Co.,Ltd.

Add: No.2 Building, Rongzhi Industry Park, No. 3667, Zhongbin Avenue, Sino-Singapore Eco-city, TEDA, Tianjin 300467, China
Tel: +86-22-25264212 **E-mail:** marketing@dynamiker.com **Web:** www.dynamiker.com