

**Dynamiker Pets Biotechnology (Tianjin) Co., Ltd.**

Canine Influenza Virus Antigen Rapid Test

Catalogue No: DNP-017

User Manual / 10 tests

**1. INTENDED USE**

The Canine Influenza Virus Antigen Rapid Test is a lateral flow immunochromatographic assay for the qualitative detection of canine Influenza virus antigen (CIV Ag) in secretions from dog's eyes, nasal cavities, or anus specimen.

Assay Time: 5-10 minutes

**2. PRINCIPLE**

The Canine Influenza Virus Antigen Rapid Test is based on sandwich lateral flow immunochromatographic assay. The test device has a testing window for the observation of assay running and result reading. The testing window has an invisible T (test) zone and a C (control) zone before testing. When the treated sample was applied into the sample well on the device, the liquid will laterally flow through the surface of the test strip and react with the pre-coated monoclonal antibodies. If there is CIV antigen in the specimen, a visible T line will appear. The C line should always appear after a sample is applied, which indicates a valid result. By this means, the device can accurately indicate the presence of Influenza virus antigen in the specimen.

**3. REAGENTS AND MATERIALS**

Components	Quantity
Test Cassette in Foil Bag	10 Tests
Desiccant in Foil Bag	10 Pcs
Disposable Dropper in Foil Bag	10 Pcs
Assay Buffer	10*1.0 mL
Cotton Tipped Applicator	10 Pcs
User Manual	1 Pc


**4. STORAGE AND STABILITY**

The kit can be stored at room temperature (4-30°C). The test kit is stable through

the expiration date marked on the package label. DO NOT FREEZE. Do not store the test kit in direct sunlight.

**5. TEST PROCEDURE**

- 5.1 Collect dog's ocular, nasal or anus secretions with the cotton tipped applicator and make the applicator wet sufficiently.
- 5.2 Insert the applicator into the provided assay buffer tube. Agitates it to get efficient sample extraction.
- 5.3 Take out the test device from the foil bag and place it horizontally.
- 5.4 Pipette the treated sample extraction from the assay buffer tube and place 3 drops into the sample well labelled "S" of the test device.
- 5.5 Interpret the result in 5-10 minutes. Result after 10 minutes is considered as invalid.

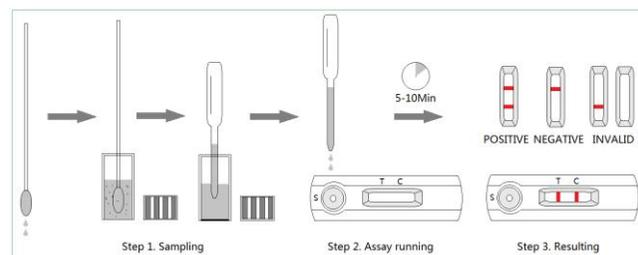

**6. INTERPRETATION OF RESULTS**

- Positive (+): Presence of both "C" line and "T" line, no matter "T" line is clear or vague.
- Negative (-): Appearance of single "C" line. No red "T" line.
- Invalid: No red "C" line appears. No matter if "T" line appears.

**7. PRECAUTIONS**

- All reagents must be at room temperature before testing.
- Do not remove test cassette from its bag until immediately before use.
- Do not use the test beyond its expiration date.
- The components in this kit have been quality control tested as standard batch unit. Do not mix components from different lot numbers.

- All specimens are of potential infection. It must be strictly treated according to the rules and regulations by local states.

**8. LIMITATION**

The Canine Influenza Virus Antigen Rapid Test is for in vitro veterinary diagnosis use only. All result should be considered with other clinical information available with veterinarian. It is suggested to apply a further confirmative method such as RT-PCR when positive result was observed.

**9. MANUFACTURER**

Company: Dynamiker Pets Biotechnology (Tianjin) Co., Ltd.  
 Address: Room 303, No.2 Building, Rongzhi Industry Park, No. 3667, Zhongbin Avenue, Sino-Singapore Eco-city, Tianjin, China  
 Post code: 300467  
 Tel: +86-022-25264212  
 Fax: +86-022-25264212

**[SYMBOLS USED]**

Symbol	Description
	Use-by date
	Lot Number
	Manufacture Date
	Keep Away from Sunlight
	Temperature Limitation

REVISED: 04/2023